

Case study: Heavily draining wounds on left leg associated with severe lymphedema

Patient: a 47-year-old morbidly obese female (580lb), suffering from numerous health issues including hypertension, sleep apnea and other co-morbidities.

Wound: heavily exuding wounds on left leg associated with severe lymphedema.

Previous treatment: the patient has been in care of wound clinicians for over 15 years and has been treated with many different products over the years with little or no effect.

Enluxtra treatment: Enluxtra-R roll treatment was initiated on May 13, 2015. The dressing was wrapped around the affected leg and secured with the provided seam tape. An additional 6"x6" Enluxtra dressing was added at the ankle to contain the drainage. Initial change frequency was every other day, then was reduced to every 3-4 days. In the course of the treatment the edema was reduced resulting in significant reduction of overall leg circumference. The treatment was completed after 10 dressing changes as the wounds' drainage rate has gone down and the skin condition improved.



